



SCULPTRA® is designed to restore a more youthful look naturally and gradually



What is SCULPTRA®?

SCULPTRA® is a biocompatible and biodegradable injectable material known as poly-L-lactic acid that naturally stimulates your cells to produce their own collagen. The result gives a more youthful and natural-looking appearance. SCULPTRA® literally refreshes the way you look by restoring your skin's natural contours and smoothing out wrinkles, creases and folds. Traditional dermal fillers only address wrinkles, lines and folds; SCULPTRA® works differently. It replenishes lost volume by stimulating growth of your own natural collagen to treat the underlying causes of facial ageing.

SCULPTRA® can restore a more youthful, natural look that lasts more than 2 years*.

**Individual results and treatment sessions may vary. Clinical trial ended at 25 months.*

How does SCULPTRA® work?

SCULPTRA® works to target the underlying causes of the signs of ageing such as collagen loss. SCULPTRA® is a facial injectable that works gradually in a series of treatments - an average of 3 injection sessions with an interval of 4 to 6 weeks between each session - to produce a refined, more youthful-looking appearance. During the course of your SCULPTRA® treatments, collagen growth is stimulated. Over time, you'll notice changes to your face that can last for more than 2 years.*

**Individual results and treatment sessions may vary. Clinical trial ended at 25 months.*

Does treatment hurt?

As with all injections you will probably feel some discomfort. You may even notice a brief burning sensation at the injection site. Applying an ice pack after treatment will help relax your skin. If necessary your Medical Aesthetics practitioner will also give you a local anaesthetic.

What about side effects?

Your Medical Aesthetics practitioner will give you special aftercare instructions. For optimum effects it is important to massage the treatment area twice daily for one week. Inform your Medical Aesthetics practitioner if any side effects occur, for example inflammation or small bumps in the treated area.

Important safety information

You should not use SCULPTRA® if you are allergic to any ingredient of the product or have a history of keloid formation or hypertrophic scarring. It should not be injected while you have an active skin infection or inflammation in the treatment area and should not be injected into the red area of the lip. Use in the skin near the eyes is not recommended. Some side effects may include injection site discomfort, redness, bruising, bleeding, itching and swelling. Other side effects may include small lumps under the skin that are sometimes noticeable when pressing on the treated area. Larger lumps, some with delayed onset with or without inflammation or skin discoloration, have also been reported. Some effects can be avoided if you follow your proper aftercare recommendations.

Physician stamp

Sinclair IS Pharma
Whitfield Court, 30-32 Whitfield Street,
London W1T 2RQ
Tel: +44 (0) 1244 625 152



UK/SIPSCU/13/0005



your secret
Turn back time and rediscover your beauty
without anyone knowing you have received
an aesthetic treatment...



